



**NEW**

The innovative padded bed for protection against impact injuries

# Regia gardo

The bed for optimum all-round protection

Diseases such as Huntington's disease, whose symptoms include uncontrolled movements, expose patients to a higher risk of impact injuries. Their beds are often individually padded, for example with foam pieces cut to size. Burmeier's Regia gardo now offers a contemporary solution that combines safety, comfort and optimum hygiene.

The Regia gardo combines all the strengths of a modern, electrically-adjustable care bed with custom-made padding on the inside. The headboard, footboard and full-length safety sides are equipped slip-on padded covers that are easy to remove for thorough cleaning. The bed has an attractive appearance and is pleasing to the touch, which makes it a place of dignified peace and relaxation.

## Strong advantages

- Low-height bed to prevent fall injuries
- Height adjustment range of 26-80 cm allows for back-friendly working with the bed
- Standard full-length safety sides offer additional protection and can be fitted with lateral padded surfaces
- Sitting position and reverse-Trendelenburg position available
- Safe 24-volt drive system
- Easy assembly of the bed and covers
- Suitable mattress always included





The intuitive handset can be used to move the Regia gardo through a height adjustment range of 26 to 80 cm.



▲ The full-length padding offers the resident optimum protection for relaxed rest periods.



▲ In the sitting position, residents can eat, read or use their laptop in bed. The cosy Pleto side table is an ideal companion.



▲ The lateral padding moves with the full-length safety side, which allows the resident to sit on the soft padded cover before getting out of bed at an ergonomic height.



▲ The Regia gardo is delivered as standard with a mattress that adapts flexibly to the adjustment positions of the mattress base.

## Functions and details



▲ The covers are fitted closely to the headboard and footboard to avoid creating unhygienic gaps.



▲ Segifix locks keep the ends of lateral covers positioned tightly against the bed frame.




▲ The pieces of padding are easy to remove for thorough cleaning.



▲ This seamless all-round protection cushions residents against impact injuries.


### Special features:

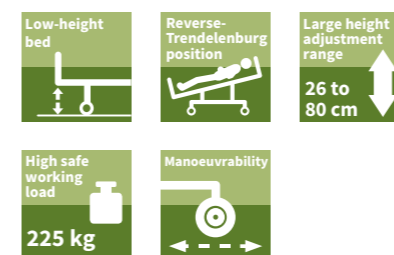

▲ The optional wall deflection roller prevents collision damage while moving the bed and adjusting its height.



▲ Convenient Velcro straps ensure that no gaps appear between the lateral covers and the bed frame.

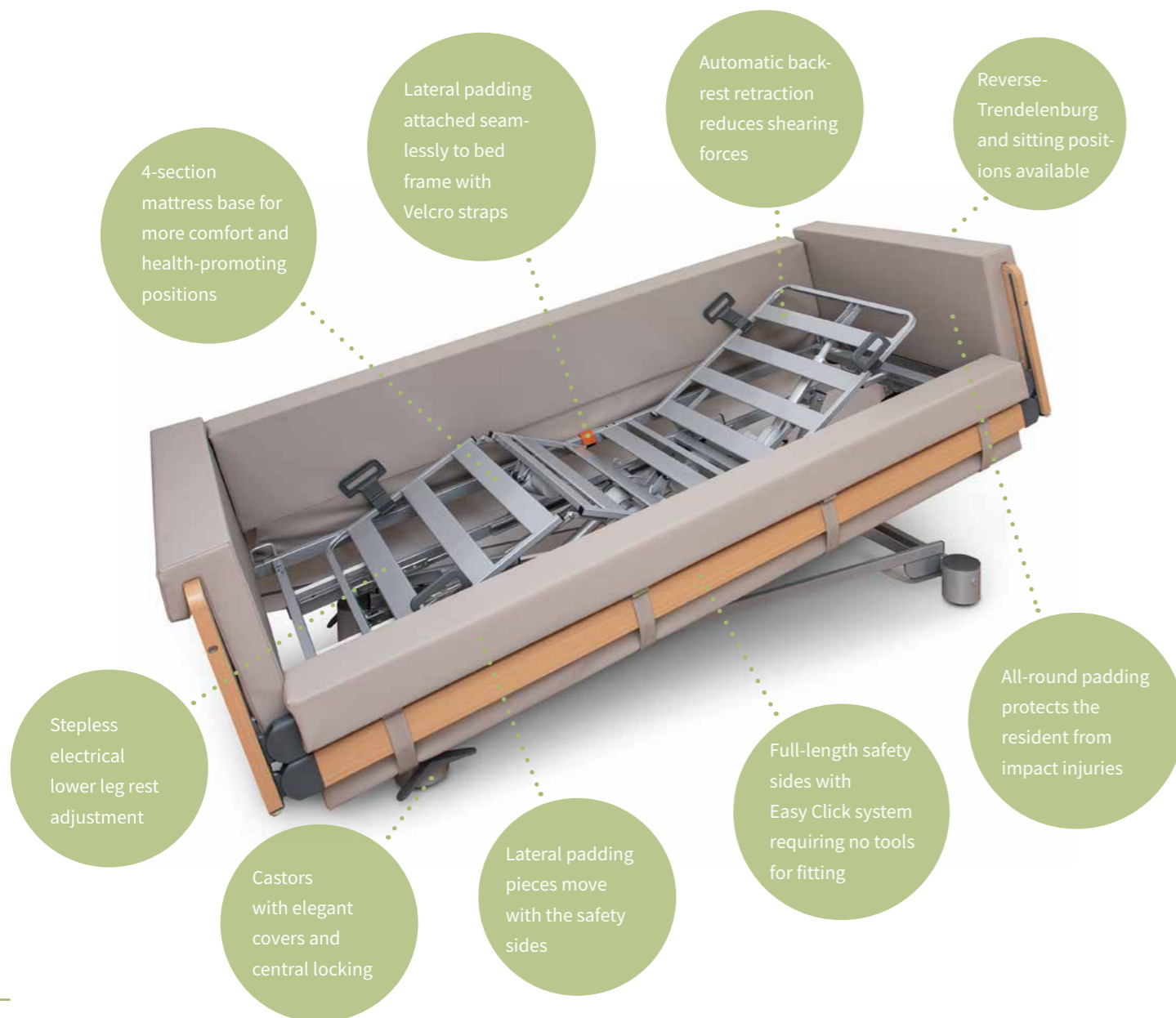


# Equipment and options

Features	Regia gardo with standard equipment: 304400
Mattress base 90 × 200 cm	✓
Mattress 90 x 209 x 14 cm	✓
Height adjustment	26 – 80 cm
Safe working load	225 kg
4-section metal slat mattress base with backrest retraction	✓
Electric raising of the lower leg rest (21 cm)	✓
Reverse-Trendelenburg position (15°)	✓
4 central locking castors	✓
Linak 24-volt drive system	✓
Nobla headboard and footboard (only wood decors)	✓
Decor Natural Beech	✓
Full-length safety side (DSG) with Easy Click on both sides	✓
CPR release by battery (item number: 275602)	O
Padding	✓
Wall deflection rollers (choice of 2 or 4 rollers Item number: 269834)	O
Trendelenburg handset (for nursing homes only – Item number: 261651)	O

✓ = Standard feature O = Available as an option – = Not possible

## External dimensions/height adjustment range:

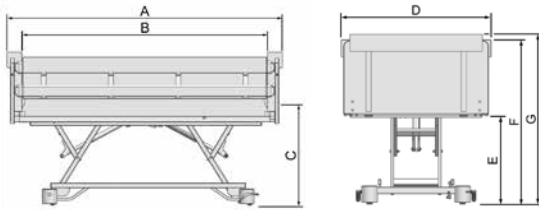

Bed length outside (with covers) A	2350
Bed length inside (with covers) B	2080
Mattress base height (to mattress base) C	260
Mattress base height (to top edge of mattress) C	400
Bed width outside (with covers) D	1130
Height of lower edge of head/footboard E	130   680
Height of safety side (with covers) F	700   1250
Height of head/footboard side (with covers) G	750   1300
Mattress base division (backrest/seat/thigh rest/ lower leg rest)	720 – 250 – 330 – 540
Backrest length compensation	110

  
**STIEGELMEYERGROUP**  
 Solutions for people who care



### Hauptsitz / Headquarters

Deutschland / Germany  
**Burmeier GmbH & Co. KG**  
 Industriestraße 53  
 32120 Hiddenhausen  
 Phone +49 (0) 5223 9769 - 0  
 Fax +49 (0) 5223 9769 - 090  
 info@burmeier.com  
 www.burmeier.com



### Hauptsitz / Headquarters

Deutschland / Germany  
**Stiegelmeier GmbH & Co. KG**  
 Ackerstraße 42  
 32051 Herford  
 Phone +49 (0) 5221 185 - 0  
 Fax +49 (0) 5221 185 - 100  
 info@stiegelmeier.com  
 www.stiegelmeier.com



stiegelmeier-group.com