

## Out-of-Bed flex


The accomodating solution  
for bed exit notifications

# Out-of-Bed flex – the sensor bar for flexible use

The retrofittable Out-of-Bed assistance system (OOB flex) notifies caregivers whenever a resident gets up and does not return to bed within a specified period of time. It offers protection and quality of life combined with cost-efficient flexibility in residential care. The OOB flex system enables nursing homes to cater appropriately to the needs of fall-prone residents. The system can be installed,

only when required, on different Stiegmeyer care beds. The OOB flex system enables nursing homes to meet a tailored investment need, and provide a high standard of care for their residents while also easing the workload of caregivers – particularly in situations with fewer people on duty.

## Easy to use and install

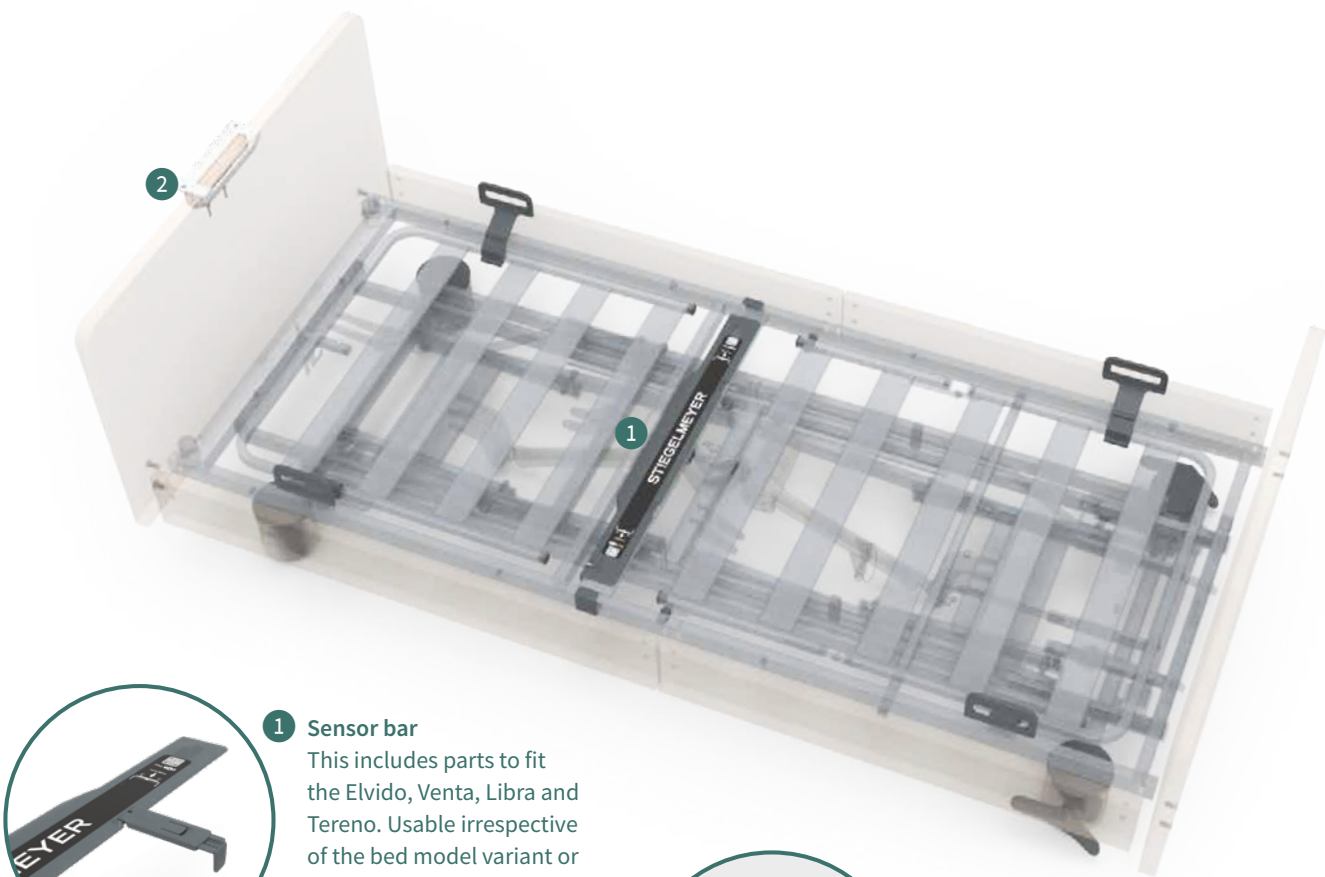

**1 Sensor bar**  
This includes parts to fit the Elvido, Venta, Libra and Tereno. Usable irrespective of the bed model variant or the mattress base width.



**2 Control unit**  
The control unit can be attached at an ergonomic place to suit the particular care situation on the headboard, footboard or side panels. The power supply can be customised either via the bed control or connected to a power socket.



**Proven operating concept**  
The user interface offers a choice of four different time settings activated at the touch of a button. Many users are already familiar with this proven and convenient concept from our integrated Out-of-Bed smart system.



**Better orientation at night**  
A reading lamp can be connected to the automatic light system provided by OOB flex, which switches the light on whenever the resident gets up. This provides better orientation at night and reduces the risk of falls.



The Out-of-Bed flex system makes work easier for caregivers in many different ways. With it's control unit placed in reach at an ergonomic working height there is no need for caregivers to look for it or reach below the bed to find it. It also provides a practical aid to decision-making thanks to the system's preset times. These time delays between under 3 seconds and 15 minutes until a notification is triggered. During day-to-day care, this alleviates

the pressure of responsibility while still providing the resident with full protection.

Activating the OOB flex system is possible even when the bed is at its lowest position of 25 cm (Tereno: 15 cm) during residents' rest periods. This makes work appreciably easier for caregivers, especially at night.

## Your advantages at a glance

- Flexible use – no design or technical requirements to be fulfilled by the Stiegmeyer care bed
- Tool-free assembly
- The range of preset times is based on the proven design of the Stiegmeyer OOB smart handset
- The power supply can be customised to take account of the structural situation either through the use of an external power supply unit or via the Linak bed control
- OOB flex can be enhanced with the addition of a reading lamp
- Flexible option for connecting to standard nurse call systems



# Technical data and compatible care bed models

## OOB flex components and features

Components	Control unit, sensor bar, power supply unit, reading lamp with direct connection cable (optional) and call system connection cable (optional)
Operating voltage	18-40V DC
Protection category	IPX4

## Compatible care beds

Stieglmeyer	<ul style="list-style-type: none"> <li>- Libra product range</li> <li>- Tereno</li> <li>- Basano</li> <li>- Elvido product range</li> </ul>	<ul style="list-style-type: none"> <li>- Venta product range</li> <li>- Amara / Amara tero</li> <li>- Arena</li> <li>- Pareno</li> </ul>
Burmeier	<ul style="list-style-type: none"> <li>- Regia</li> <li>- Verano</li> <li>- Lenus</li> <li>- Dali with metal mattress base</li> </ul>	<ul style="list-style-type: none"> <li>- Dali economic</li> <li>- Westfalia-Klassik</li> <li>- Gigant</li> <li>- Inovia</li> </ul>



### Hauptsitz / Headquarters

Deutschland / Germany  
**Stieglmeyer GmbH & Co. KG**  
 Ackerstraße 42  
 32051 Herford  
 Phone +49 (0) 5221 185 - 0  
 Fax +49 (0) 5221 185 - 100  
 info@stieglmeyer.com  
 www.stieglmeyer.com



### Hauptsitz / Headquarters

Deutschland / Germany  
**Burmeier GmbH & Co. KG**  
 Industriestraße 53  
 32120 Hiddenhausen  
 Phone +49 (0) 5223 9769 - 0  
 Fax +49 (0) 5223 9769 - 090  
 info@burmeier.com  
 www.burmeier.com

France  
**Stieglmeyer SAS**  
 16 Avenue de l'Europe  
 Bâtiment SXB1  
 67300 Schiltigheim  
 Phone +33 (0) 4 7238 - 5336  
 contact@stieglmeyer.fr  
 www.stieglmeyer.fr

Nederland / Netherlands  
**Stieglmeyer BV**  
 Peppelenbos 7  
 6662 WB Elst  
 Phone +31 (0) 345 581 - 881  
 info@stieglmeyer.nl  
 www.stieglmeyer.nl

Polska / Poland  
**Stieglmeyer Sp. z o.o.**  
 86-212 Stolno  
 Grubno 63  
 Phone +48 56 6771 - 400  
 Fax +48 56 6771 - 410  
 info.stolno@stieglmeyer.pl  
 www.stieglmeyer.pl

China  
**Stieglmeyer China Co. Ltd.**  
 Poly Center, No. 8 South Wenhua Road  
 Dongping New Town, Shunde District  
 528300 Foshan  
 Phone +86 757 29878710 - 0  
 info@stieglmeyer.cn  
 www.stieglmeyer.cn

South Africa  
**Stieglmeyer Africa (PTY) Ltd.**  
 5 Shiraz Street, Oude Westhof  
 Bellville, 7530  
 Phone +27 (0) 21 555 - 1415  
 Fax +27 (0) 21 555 - 0639  
 info.southafrica@stieglmeyer.com  
 www.stieglmeyer.co.za

Suomi / Finland  
**Stieglmeyer Oy**  
 Paanakatu 5  
 15150 Lahti  
 Phone +358 (0) 3 7536 - 320  
 stieglmeyer@stieglmeyer.fi  
 www.stieglmeyer.fi

België / Belgique / Belgium  
**Distrac Group**  
 Bleyveld 14  
 3320 Hoegaarden  
 Phone +32 (0) 16 76 82 82  
 info@distracgroup.com  
 www.distracgroup.com



stieglmeyer-group.com