



Out-of-Bed flex


The accommodating solution
for bed exit notifications

Out-of-Bed flex – the sensor bar for flexible use

The retrofitable Out-of-Bed assistance system (OOB flex) notifies caregivers whenever a resident gets up and does not return to bed within a specified period of time. It offers protection and quality of life combined with cost-efficient flexibility in residential care. The OOB flex enables nursing homes to respond to the individual needs of their residents as required. The system can be installed,

only when required, on different Stiegmeyer care beds. The OOB flex system enables nursing homes to meet a tailored investment need, and provide a high standard of care for their residents while also easing the workload of caregivers – particularly in situations with fewer people on duty.

Easy to use and install

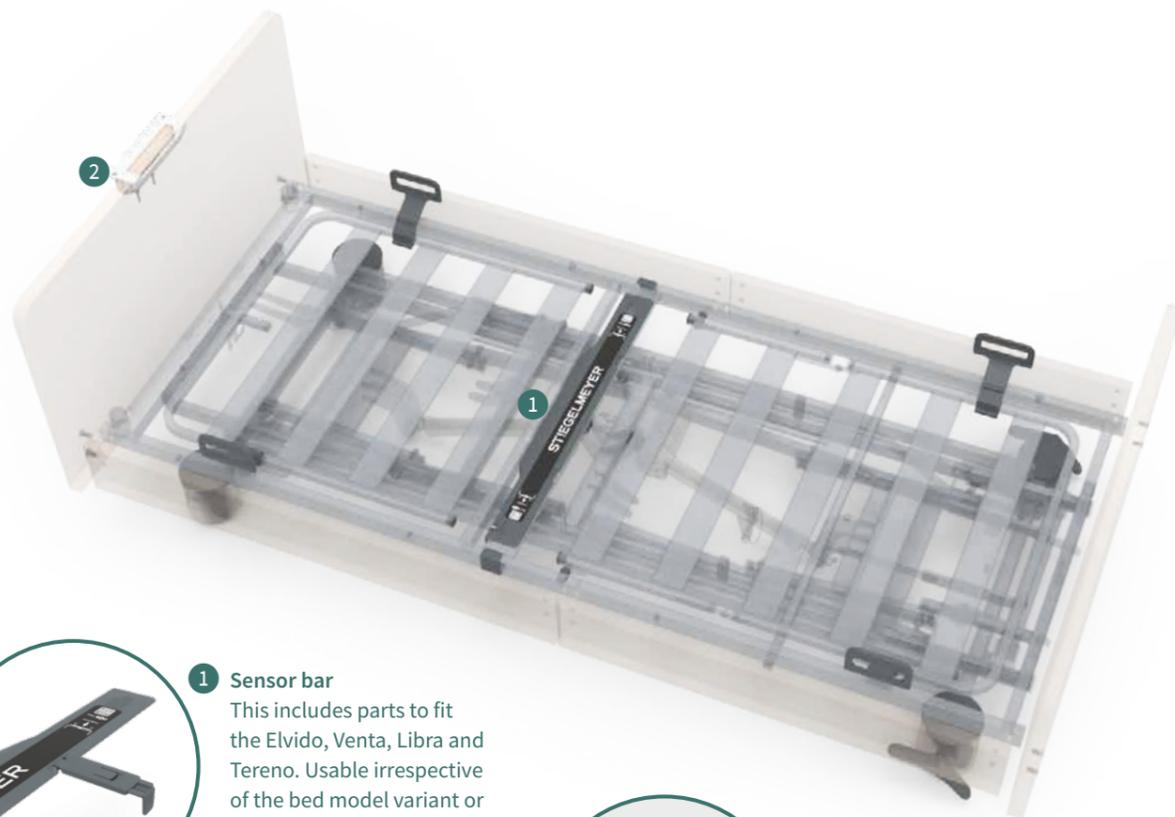

1 Sensor bar
This includes parts to fit the Elvido, Venta, Libra and Tereno. Usable irrespective of the bed model variant or the mattress base width.



2 Control unit
The control unit can be attached at an ergonomic place to suit the particular care situation on the headboard, footboard or side panels. The power supply can be customised either via the bed control or connected to a power socket.



Proven operating concept
The user interface offers a choice of four different time settings activated at the touch of a button. Many users are already familiar with this proven and convenient concept from our integrated Out-of-Bed smart system.



Better orientation at night
A reading lamp can be connected to the automatic light system provided by OOB flex, which switches the light on whenever the resident gets up. This provides better orientation at night and increases safety.



The Out-of-Bed flex system makes work easier for caregivers in many different ways. With its control unit placed in reach at an ergonomic working height there is no need for caregivers to look for it or reach below the bed to find it. It also provides a practical aid to decision-making thanks to the system's preset times. These time delays between under 3 seconds and 15 minutes until a notification is triggered. During day-to-day care, this alleviates

the pressure of responsibility and also increases the safety of residents.

Activating the OOB flex system is possible even when the bed is at its lowest position of 25 cm (Tereno: 15 cm) during residents' rest periods. This makes work appreciably easier for caregivers, especially at night.

Your advantages at a glance

- Flexible use – no design or technical requirements to be fulfilled by the Stiegmeyer care bed
- Tool-free assembly
- The range of preset times is based on the proven design of the Stiegmeyer OOB smart handset
- The power supply can be customised to take account of the structural situation either through the use of an external power supply unit or via the Linak bed control
- OOB flex can be enhanced with the addition of a reading lamp
- Flexible option for connecting to standard nurse call systems



Technical data and compatible care bed models

OOB flex components and features

Components	Control unit, sensor bar, power supply unit, reading lamp with direct connection cable (optional) and call system connection cable (optional)
Operating voltage	18-40V DC
Protection category	IPX4

Compatible care beds

Stieglmeyer	<ul style="list-style-type: none"> - Libra product range - Tereno - Basano - Elvido product range 	<ul style="list-style-type: none"> - Venta product range - Amara / Amara tero - Arena - Pareno
Burmeier	<ul style="list-style-type: none"> - Regia - Verano - Lenus - Dali with metal mattress base 	<ul style="list-style-type: none"> - Dali economic - Westfalia-Klassik - Gigant - Inovia



Hauptsitz / Headquarters

Deutschland / Germany
Stieglmeyer GmbH & Co. KG
 Ackerstraße 42
 32051 Herford
 Phone +49 (0) 5221 185 - 0
 Fax +49 (0) 5221 185 - 100
 info@stieglmeyer.com
 www.stieglmeyer.com



Hauptsitz / Headquarters

Deutschland / Germany
Burmeier GmbH & Co. KG
 Industriestraße 53
 32120 Hiddenhausen
 Phone +49 (0) 5223 9769 - 0
 Fax +49 (0) 5223 9769 - 090
 info@burmeier.com
 www.burmeier.com

France

Stieglmeyer SAS
 16 Avenue de l'Europe
 Bâtiment SXB1
 67300 Schiltigheim
 Phone +33 (0) 4 7238 - 5336
 contact@stieglmeyer.fr
 www.stieglmeyer.fr

Nederland / Netherlands

Stieglmeyer BV
 Peppelenbos 7
 6662 WB Elst
 Phone +31 (0) 345 581 - 881
 info@stieglmeyer.nl
 www.stieglmeyer.nl

Polska / Poland

Stieglmeyer Sp. z o.o.
 86-212 Stolno
 Grubno 63
 Phone +48 56 6771 - 400
 Fax +48 56 6771 - 410
 info.stolno@stieglmeyer.pl
 www.stieglmeyer.pl

China

Stieglmeyer China Co. Ltd.
 Poly Center, No. 8 South Wenhua Road
 Dongping New Town, Shunde District
 528300 Foshan
 Phone +86 757 29878710 - 0
 info@stieglmeyer.cn
 www.stieglmeyer.cn

South Africa

Stieglmeyer Africa (PTY) Ltd.
 5 Shiraz Street, Oude Westhof
 Bellville, 7530
 Phone +27 (0) 21 555 - 1415
 Fax +27 (0) 21 555 - 0639
 info.southafrica@stieglmeyer.com
 www.stieglmeyer.co.za

Suomi / Finland

Stieglmeyer Oy
 Paanakatu 5
 15150 Lahti
 Phone +358 (0) 3 7536 - 320
 stieglmeyer@stieglmeyer.fi
 www.stieglmeyer.fi

België / Belgique / Belgium

Distrac Group
 Bleyveld 14
 3320 Hoegaarden
 Phone +32 (0) 16 76 82 82
 info@distracgroup.com
 www.distracgroup.com



stieglmeyer-group.com